


FACEPROTECT2

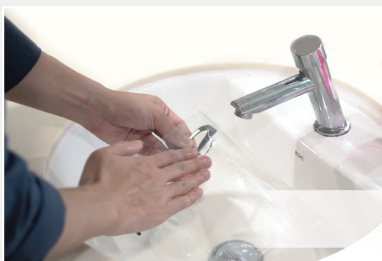

Use

- 1 Wash hands with soap and water
- 2 Assemble face shield
- 3 Adjust securing strap to preferred length
- 4 Don the face shield

Wash and Reuse

- 1 After each use, wash face shield with soap and water
- 2 Wipe dry and air face shield
- 3 Store in a cool and dry place before next use

**Dismantling before wash is recommended*

Designed by:



Manufactured by:



Components




Forehead Strap x 1



Forehead Pad x 1



Securing Strap x 1

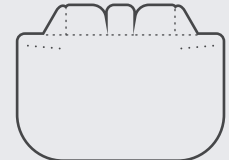

Visor x 1




Adaptor (Left) x 1




Adaptor (Right) x 1

1

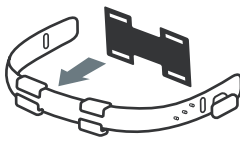


Position forehead pad behind forehead strap.

2

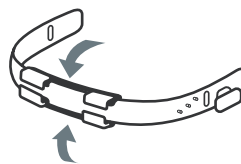


Slot the four holes of forehead pad into the four ridges of forehead strap.

3

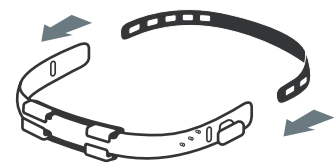


Position securing strap onto hooks on both sides of forehead strap. Adjust length to preferred fit.

4

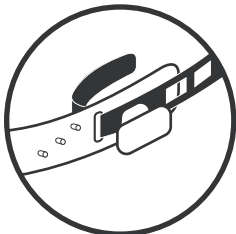


Trim or tuck in excess strap.

5

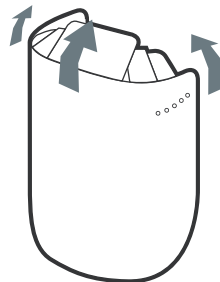


Gently fold visor's top flap to ~45 degrees inwards.

6

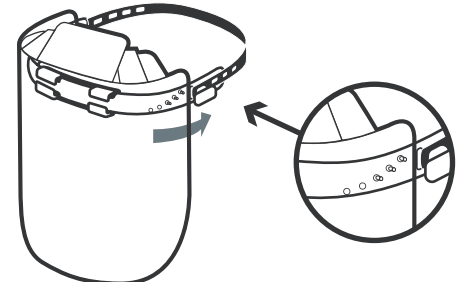


Snap visor onto assembled headstrap.


Adjust Visor (3 Options)

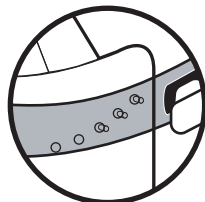

Attach to outer 3 holes.

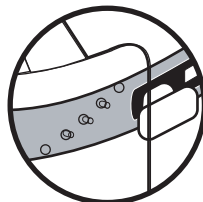

Attach to middle 3 holes.

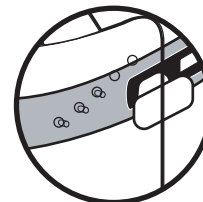


Attach to inner 3 holes.

Designed by:

Manufactured by:


Attaching the adaptor for flip-up visor (Optional)

1

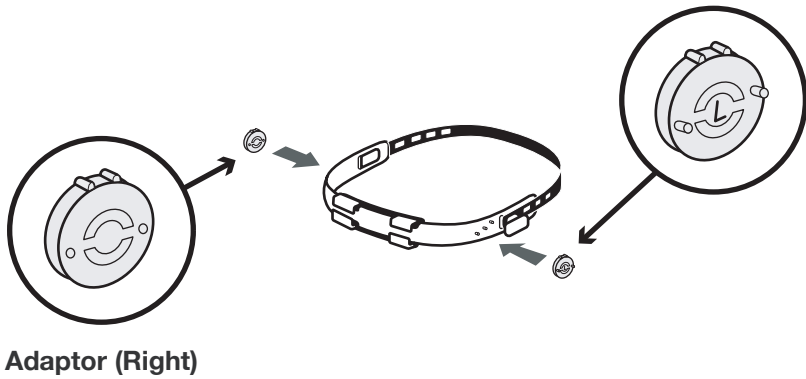


Snap both adaptors onto assembled forehead strap.

2

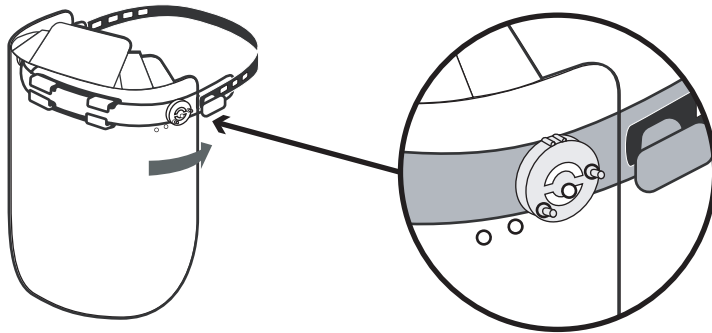


Snap visor on top of adaptors.
It is recommended to attach the visor to the outer 3 holes.


3

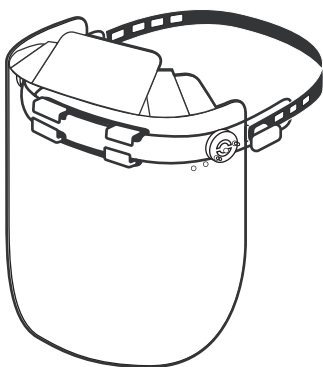


Visor for protection.

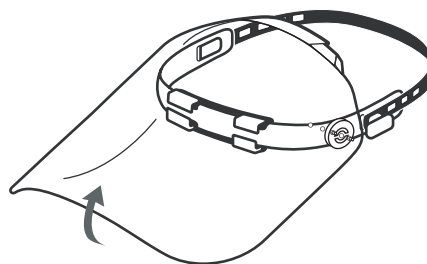


Visor in flip-up position.

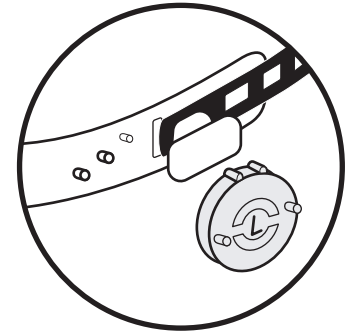

Snap adaptor onto front 2 holes of the forehead strap.

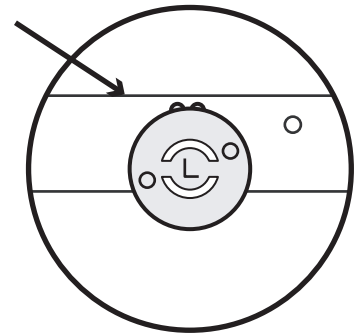

Ensure accurate position of adaptors.

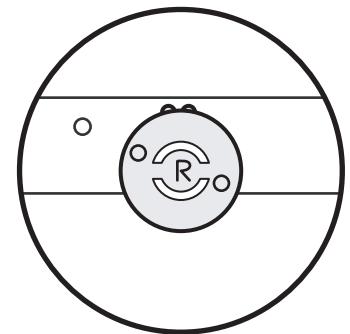

"L" and "R" markings should be upright.

Designed by:

Manufactured by: